WATER SPORTS

SURF
KITE
WINGFOIL
WINDSURF
SWIM
...

HYDROPAD®

SAS au capital de 100 000€ RCS Montpellier
501115695002F-34160 CASTRIES - FRANCE

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Protect your ears from water and cold, keep hearing and balance.

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HOW TO USE

To insert your HYDROPADS into your ear canal, lift your pinna slightly and insert the tip into the ear with your other hand until it fits snugly. It is very easy to remove thanks to the cord or body of the filter. Improper insertion, misuse, or failure to follow directions will limit effectiveness.

Do not insert the earplug too deeply into the ear. Before each use, check the condition of the tips, the shell and/or the slightest visual defect that could alter the effectiveness of the protectors.

Please store and transport your protectors in their protective case before and after use, away from excessive heat.

Regularly clean your HYDROPAD. Salt crystals can form on the filter and affect its efficiency. In order to restore the sound performance of your HYDROPAD, rinse them thoroughly with fresh, lukewarm water and then let them dry.

We recommend using the lanyard to reduce the risk of loss. Before using the product, try the different sizes of tips to ensure the

INFORMATIONS

The average lifespan of SLEEPAD protectors is 3 years from the first use. It also depends on the frequency of use of your protections. If you notice a drop in filter efficiency (too strong attenuation) or otherwise, replace your HYDROPAD.